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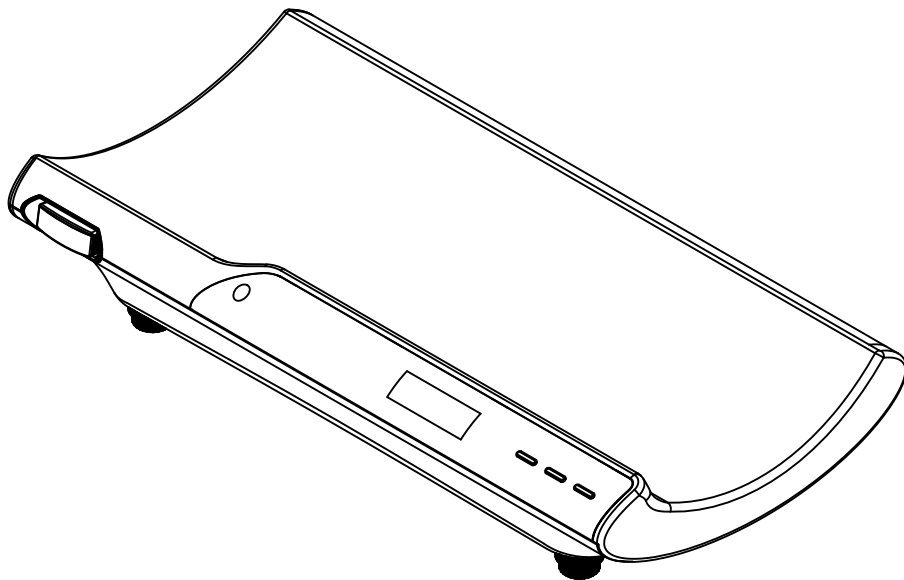
After Sales Service : Terrailon France & Headquarters - B.P.73 -78403 Chatou Cedex-France

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FOOK TIN TECHNOLOGIES LTD

4/F, Eastern Centre, 1065 King Road, HONG KONG

www.fooktin.com.hk

CLASS III ELECTRONIC MEDICAL BABYSCALE

Dear Customer,

Thank you for purchasing this product. We hope you will find it satisfactory.

Please read these instructions carefully to take full advantage of your product.

READ BEFORE USE

The Electronic Medical Babyscale is designed and manufactured in a facility certified in the following Management Systems : ISO 9001 Quality, ISO14001 Environment, OHSAS18001 Health and Safety and ISO13485 Medical Devices Quality.

Applications

The electronic medical babyscale's intended use is for weighing patients for the purposes of monitoring, diagnosis and medical treatment.

Precautions of use

		This symbol indicates general precautions that should be taken when using this product.
	WARNING	This symbol indicates the possibility of serious injury if the product is mishandled or instructions are ignored.
	CAUTION	This symbol indicates the possibility of physical injury or equipment damage if instructions are ignored.

- Do not reconnect power until the LCD is completely powered off.
- Use the wheels located in the bottom to move the scale around.

Warning

- Remove all weight from the platform before measurement.
- Use the product on a firm and flat surface.
- Clean after use with a lightly dampened cloth without allowing any water to get into the device. Do not use solvents or immerse the product in water.
- Avoid excessive impact or vibration to the unit, such as dropping the product onto the floor.
- Do not jump on the weighing platform.
- Only use the adaptor which has the characteristics mentioned in the paragraph: Product Specification.

Caution

- Do not disassemble the product as incorrect handling may cause injury.

MAINTENANCE

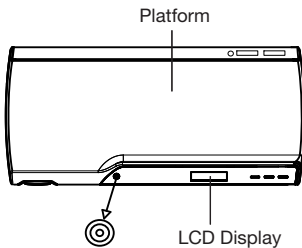
- The product has been manufactured and calibrated with precision. The user should not disassemble or attempt to calibrate the product.
- Remove batteries if the product is not in use for an extended period.

CLASS III ELECTRONIC MEDICAL BABYSCALE


PRODUCT SPECIFICATION

- Class I medical device (EC Directive 93/42 EEC).
- Class III OIML compliant.
- Class III Non-Automatic Weighing Instruments (EC Directive 90/384 EEC) compliant.
- Weighing function with tare.
- Level indicator and adjusting feet.
- Auto-off function if product operates with batteries. In case the AC/DC adaptor is plugged in, product will stay on power until the [Off] key is pressed down.
- Low-battery indicator.
- Capacity: 15 kg.
- Graduation: 5 g (0 – 7.5 kg) / 10 g (7.5 – 15 kg).
- Operates through 6 pieces AA batteries included or with AC 100-240V adaptor input/DC 9V adaptor output (adaptor included).
- Measuring tape for height and head circumference included

Front View


Back View


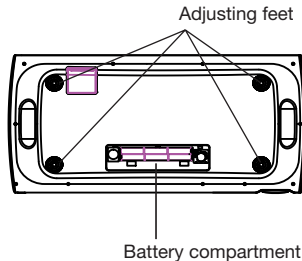
PRODUCT USAGE

Initialisation

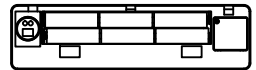
- Insert the battery(s) in the battery compartment, making sure that the polarities are correct or power up the product by connecting the DC adaptor to the power socket. The battery type is specified inside the battery compartment.
- When batteries are changed, product will turn off automatically even if the adaptor is plugged in.
- It is necessary to set the balance of the product before you use it for the first time or if the product has been moved to another place:

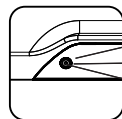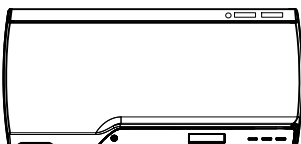1. Remove all weight from the platform.
2. Place the product on a hard and flat surface. Use the adjusting feet to adjust the balance until the bubble inside the level indicator is located in the middle of the drawn circle.

Level indicator
Circle
Bubble

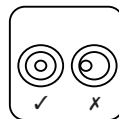CLASS III ELECTRONIC MEDICAL BABYSCALE

Operation

1. Press [On] key to turn the product on. The following displays will appear in the following order:

- When ~ and +0+ icons appear on the display, place the baby on the product. The weight will appear automatically. The weight is stable when ~ icon appears once again on the LCD display.
- If the product operates with battery and is not used for 20 seconds, it will turn off automatically. If the product operates on main, press [Off] key to turn it off.

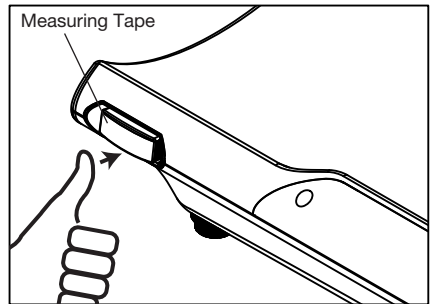
Weighing with Tare

- In weighing mode, press [Tare] key. « NET » will keep on flashing unless stable tare value has been obtained.
- When  appears and when the « NET » indication is not flashing anymore, the product is ready for the next measurement.

Measuring Tape usage

When the babyscale is turned on and when the measuring tape is plugged off the product, the babyscale will then turn off automatically even if the adaptor is plugged in.

If the measuring tape is inserted again in the product, the babyscale will turn on automatically when the adaptor is being used. If the product is operating with batteries, [On] key will need to be pressed down to turn on the product.

Warning indicators

Err0	Initial zero error. Step off the scale, wait until the scale shows 0.0000 and repeat measurement	bAt	Low battery indication: replace it.
Ustb	Unstable zero. Please wait until the scale shows 0.000 and repeat measurement.	AdErr	Hardware problem. Please contact place of purchase.
Err2	Overload: remove weight immediately.	EEP	Hardware problem. Please contact place of purchase.
Err3	Hardware problem. Please contact place of purchase.		

CLASS III ELECTRONIC MEDICAL BABYSCALE

PROTECT THE ENVIRONMENT

Dispose of used batteries in a designed container so that they can be collected and recycled. Do not mix different types of batteries. Do not mix new and used batteries.

After life, dispose of this product at a designated waste recycling point.

GUARANTEE

This product is guaranteed against defects in materials or manufacturing. During the guarantee period, any such defects will be repaired free of charge (proof of purchase must be shown in the case of a complaint under guarantee). The guarantee does not cover damage resulting from improper use or negligence. In the event of a claim, first contact the shop where you purchased the product.

AFTER SALES SERVICE

For calibration and after sales services, please contact Terraillon consumer service at the following address:

After Sales Service: Terraillon France & Headquarters
B.P.73 -78403 Chatou Cedex-France
Service consommateurs :0 826 88 1789
serviceconsommateurs@terraillon.fr
www.terraillon.com

FOOK TIN TECHNOLOGIES LTD

4/F, Eastern Centre, 1065 King's Road, HONG KONG
www.fooktin.com.hk

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Viale Sarca 45
Milano - Italy
Tel: +39 (0)2 66114199

Terraillon Asia Pacific Ltd

4/F, Eastern Centre
1065 King's Road
Quarry Bay - Hong Kong
Tel : +852 (0)2960 7200
customerservice@terraillon-asia.com

Terraillon Corp

3737 Glenwood Ave, Ste 100
Raleigh, NC, USA 27612
Tel: 1-919-573-6039
customerserviceusa@terraillon.com

Terraillon
PROFESSIONAL
www.terraillon.com

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Hanson (UK) Ltd
Terraillon Deutschland GmbH
Terraillon SpA